

# Fitting Your Regulator and Flowmeter to a Cylinder

1. Ensure the cylinder is firmly secured.



2. Remove protective plastic cover and Inlet/Outlet plugs from Cylinder head.



3. Align the guiding pins located on the regulator to the corresponding holes on the cylinder head.



4. Secure the regulator to the cylinder by tightening the wing screw.



5. Attach the Flowmeter inlet to the Regulator Outlet by securing the hand wheel.



6. a) C Sized Cylinder - Using a cylinder key spanner, **Slowly** and fully open the cylinder valve.



- b) D, E, G Cylinder - Using The Cylinder valve, **Slowly** and fully open the cylinder valve.

