


Operating Your Oxygen Concentrator

1. Ensure the power switch is in the OFF position.



2. Connect the oxygen administration tube/Nasal Cannula onto the concentrator.

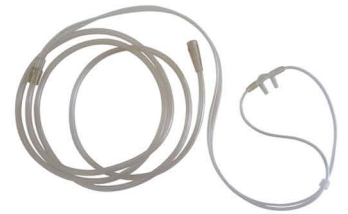

3. Ensure that all of the parts are connected securely to avoid leaks.



4. Plug the power cable into a mains socket.



5. Press the power switch to the start position. An alarm operating test is carried out automatically when the machine is switched on (this test lasts about 5 seconds).



Operating Your Oxygen Concentrator

6. Turn the flow adjustment knob to the Litres Per Minute (LPM) setting as prescribed.




7. Wait 3-5 Minutes for the orange light to turn off.



8. Check that the oxygen flows out of the administration device.

9. Adjust the nasal cannula accordingly on the patient.

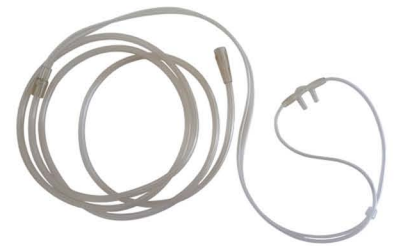

Please Note: optimal oxygen concentration is obtained about ten minutes after the device is switched on (90% of the concentration is obtained after around 5 minutes)

10. Upon completion of the treatment, turn the power switch off and unplug the power cable from the mains socket.

